


EUROPROBE

Quick start guide of the Gamma module in 3 steps



1 – Plug the power cord in the socket at the back of the module

2 - Connect the probe(s) before switching the module on, with the connector mark  up **

Automatic recognition of probes. After switching on the module, the green LED will indicate which probe is in operation



3 - Switch on the device by pressing the switch button. By Default: Tc99 selected, Sensitivity scale set to 1000, integration time set to 1 second, sound set to BIP

Adjusting sound volume: Press the knob to turn the LED on, and turn to desired level.

Setting sensitivity scale: Press the knob to turn the LED on, and turn to desired level

Example for 50 MBq of Tc99:

Injection the day before: Set sound scale at 100 or 250

Injection in the morning: Set sound scale at 500 or 1000

Set-up choices before or during use

Isotope selection: Tc99 - Press to select another isotope

Counting time: 10" - Press to select another time variable

Integration time: 1" - Press to select another integration time

Select config: Press to select the configuration (No. 0 by default)

Sound mode: BIP - Press to select FM



WARNING: TURN OFF DEVICE BEFORE CONNECTING OR DISCONNECTING PROBES

Warning: This Quick Start Guide* is not substitute to the user manual.

EUROMEDICAL
INSTRUMENTS

EUROPROBE

Getting Started with the Fluorescence Module



1 - Place the Gamma module on the Fluorescence module -
Fix the Velcro then connect the two modules using the cable according to the color code

2 - Connect the DUAL Gamma Fluo probe before start-up**



3 - Connect the pedal to the device 

4 - Switch on the device by pressing the button. Default: Tc99



Use: Switch from Gamma detection (Tc99) to Fluorescence detection (Opto) by pressing on the pedal for 3 seconds.

Contacts

Sales contact

EuroMedical Instruments
BP 241 – 3 rue de l'arrivée
Office 1018 bis
75749 Paris Cedex 15- France
Tel : + 33 9 83 62 63 17
Fax : + 33 9 83 82 63 17
sales@em-instruments.com
www.em-instruments.com

Manufacturer

EuroRad
2 rue Ettore Bugatti
67201 Eckbolsheim - France
Tel : +33 3 88 26 81 30
Fax : +33 3 88 28 45 48
www.euroRad.com
info@euroRad.com



EUROMEDICAL
INSTRUMENTS

**All probes must be disinfected and placed in a sterile sleeve before use